

The Digital Pilot Test

The Digital Pilot Test is an audiometer unlike any other. Now you can easily screen children as young as three years old and be confident of getting accurate, reliable results.



- Select Picture Audiometry with enhanced digital sound clarity
- CPT code reimbursement (CPT code 92583)
- Test English or Spanish speaking children
- Use the manual pure-tone audiometer for older children
- Large LCD display screen
- Optional printer

Select Picture Audiometry

The Digital Pilot Test screens children as young as three years old by presenting the hearing test as an interactive game. As a game, the test is easy for children to understand. As a test, the procedure gives you unmistakable responses for accurate results. You can even see the test progress on the LCD screen.

English or Spanish Word Lists

- 10 sentences presented at descending decibel levels

ENGLISH

"POINT TO THE BATHTUB."

(50 dB)

"SHOW ME THE AIRPLANE."

(40 dB)

"WHERE IS THE HOTDOG?"

(30 dB)

"POINT TO THE SAILBOAT."

(20 dB)

"SHOW ME THE CUPCAKE."

(15 dB)

OR

SPANISH

"MUÉSTRAME EL ANILLO."

(50 dB)

"DÓNDE ESTÁ LA CABEZA."

(40 dB)

"INDÍCAME EL CORREO."

(30 dB)

"MUÉSTRAME LA MUÑECA."

(20 dB)

"DÓNDE ESTÁ EL SOMBRERO."

(15 dB)

The Hearing Test Children Love to Take

Digital Pilot Test

It's as simple as looking at the pictures. The child listens to 10 sentences presented at descending decibel levels. After each sentence, the child points to the corresponding picture on the picture board, and you record the result.

Pure Tone Audiometry

The Digital Pilot Test is also a complete pure-tone audiometer which can be used to screen older children. With manual screening, you can choose the speed at which you present the tones. Easy fingertip operation gives you the flexibility to choose the frequency and hearing levels. Results can be recorded on the included score pads.

Standard Accessories

- TDH headphones
- Monitor phone
- Two large picture boards (one English, one Spanish)
- Two small picture boards (one English, one Spanish)
- Two score pads (one English, one Spanish)
- Operation manual
- One year limited warranty

Optional Printer

While the Digital Pilot Test includes a scorepad to manually keep track of the test results, you also have the option of purchasing the Pilot printer. The printer is connected by just one cable to the



Pilot Test and is small enough to hold in your hand. Print select picture audiometry results in seconds and you can choose between numerical data or picture format (shows each picture and its result). When performing an audiogram, the test can be printed in table or audiogram format.

Technical Information

Pure tone frequencies: 250, 500, 750, 1000, 1500, 2000, 3000, 4000, 6000, 8000 Hz

Pure tone decibel range: adjustable in 5 dB steps, -10 dBHL to 100 dBHL

Speech decibel range: 15, 20, 25, 30, 35, 40, 45, 50 dBHL

CPT Codes

92583_____Select Picture Audiometry

92552_____Pure tone audiometry (threshold); air only

Specifications subject to change without notice

ONE YEAR LIMITED WARRANTY: This warranty is extended to the original purchaser of the instrument, by Maico, through the Distributor from whom it was purchased. The warranty covers defects in material and workmanship for a period of one year from date of delivery of the instrument to the original purchaser. Accessories which are purchased from Maico at the same time as the instrument are warranted for one year from the date of purchase. For additional information contact your Maico Distributor.

