





# The ABC's of Propaq LT

<p><b>A</b></p> <p>Age Ranges</p>	<p><b>Neonate:</b> Term birth (37 weeks) through 28 days OR up to 44 weeks gestational age  <b>Pediatric:</b> 29 days to 12 years of age  <b>Adult:</b> 13 years of age and older</p>
<p><b>B</b></p> <p>Battery Info</p>	<p>Uses a Lithium Ion battery  Duration:  <ul style="list-style-type: none"> <li>ECG/Resp only: ~24 hours</li> <li>ECG/Resp/SpO2/BP every 15 minutes: 8hours</li> </ul> Durations are in Standalone AND Acuity connected, continuous monitoring, and minimal display use (defined as 5 minutes display use per hour).</p>
<p><b>C</b></p> <p>Cleaning</p>	<p><u>Device</u>  Wipe with a nearly-dry cloth moistened with an approved cleaning solution:  <ul style="list-style-type: none"> <li>Warm water, liquid soap, Coverage®, Windex®, Ovation®, Hydrogen peroxide, Wex-cide®, T.B.Q.®</li> </ul> After cleaning, remove excess solution with a soft cloth dampened with clean water, then dry. Do not let solution get into connector openings or crevices. Do not autoclave or immerse.</p> <p><u>Accessories</u>  Consult manufacturer's instructions.</p>
<p><b>D</b></p> <p>Defaults</p>	<p>Create default settings using the PC Configurator Tool.</p>
<p><b>E</b></p> <p>End Monitoring</p>	<p><u>Standby/Temporary Disconnect</u></p> <ol style="list-style-type: none"> <li>Remove cables/leads from device.</li> <li>Acknowledge equipment alert by pressing Select .</li> <li>At Acuity, the patient's waveform tile will turn <b>BLUE</b> indicating 'Standby'.</li> <li>To resume monitoring, use good lead prep/placement to apply leads, and then plug cables back into top of device.</li> </ol> <p><u>Power down/Permanent Disconnect</u></p> <ol style="list-style-type: none"> <li>Press the power button on the top right hand side of the monitor.</li> <li>When prompted, press  to select "Delete and Shut Down."</li> <li>Remove leads from patient.</li> </ol>