


Medical Device Depot Inc.

www.medicaldevicedepot.com
1-877-646-3300

Propaq® LT Vital Signs Monitor Quick Reference

Neonate — Birth through 28 days, or 44 weeks gestational
 Pediatric — 29 days through 12 years
 Adult — 13 years and older

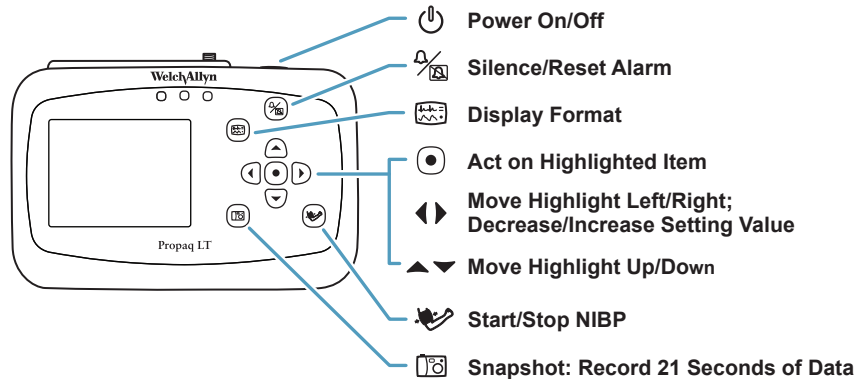

Change Alarm Limits (select) (select parameter) (change limits)

(HR/PR, NIBP, SpO₂, Resp) _____

Change Alarm Tone (SpO₂) Setup Alarms - Alarm Tone (select)

(Low, Medium, High) _____

Silence/Reset Alarm Tone

Suspend Alarm Tones (HR/min) Setup Alarms - Suspend Audible Alarms

(select)

Change (Vital Sign) Settings (select) (parameter)

(HR/PR, NIBP, SpO₂, Resp) _____

Change Patient Mode Adult (select)

(Adult, Pediatric, Neonate) _____

Change Lead II (select)

(I, II, III, ...) _____

Set NIBP mode NIBP mmHg NIBP Mode (select)

(Auto, Manual, Turbo) _____

Change Monitor Setup (NIBP) Setup (select) (set values)

(Alarms, ECG, NIBP, ...) _____

Change time/date xx:xx:xx (select) (change)

(Hour, Minute, Month, ...) _____

View Trends (NIBP) Trends

View Snapshots (HR/min) Snapshots