
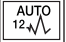

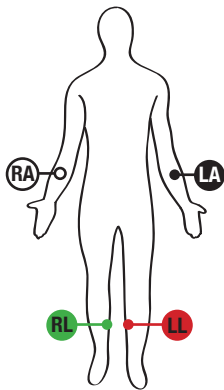





Quick Reference Guide

- 1 Prepare the Patient for the test. **NOTE:** *Proper patient preparation is essential for minimized artifact, a clean ECG and accurate results.*
 - a. **Shave** hair (if necessary) from electrode sites. (Refer to the back side of this card for proper placement.)
 - b. **Wash** area with warm, soapy water (do not use alcohol).
 - c. **Dry** skin vigorously with an abrasive pad such as a 2 x 2 or 4 x 4 gauze to remove dead skin cells and oil, and to increase capillary blood flow.
- 2 **Attach** the electrodes. (Refer to the back side of this card for proper placement.)
- 3 **Select** F1 (ID) on the IQclassic.
For **STAT ECG** – Press Auto ECG  twice.
- 4 **Enter** patient demographics:
 - a. Last Name
 - b. First Name
 - c. Patient ID
 - d. Date of Birth
 - e. Gender (M or F)
- 5 **Press** F6 (Done)
- 6 **Instruct** the patient to relax in a supine position.
- 7 **Review** the ECG on the display. Determine when the signals are of acceptable quality.
- 8 **Press** Auto ECG  or Rhythm Strip  button.
- 9 **Press** F4 Print (if ECG does not automatically print due to configuration)
- 10 **Press** Done.



Limb lead placement

- RA (White/White)** – Right deltoid, forearm or wrist.
- LA (Black/White)** – Left deltoid, forearm or wrist.
- RL (Green/White)** – Inside calf, midway between knee and ankle.
- LL (Red/White)** – Inside calf, midway between knee and ankle.

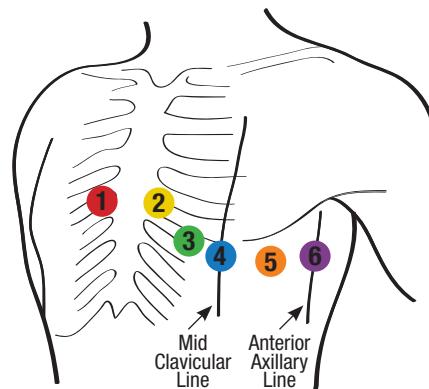

- 1 V1 (Red/Brown)** – Fourth intercostal place at the right margin of the sternum.
- 2 V2 (Yellow/Brown)** – Fourth intercostal place at the left margin of the sternum.
- 3 V3 (Green/Brown)** – Midway between V2 and V4 (on top of the 5th rib).
- 4 V4 (Blue/Brown)** – Fifth intercostal place at the left midclavicular line.
- 5 V5 (Orange/Brown)** – At the horizontal level of V4, at the left anterior axillary line.
- 6 V6 (Purple/Brown)** – At the horizontal level of V4, at the midaxillary line.